

Clinical Procedure for MAGFIT-IP



01 Implantation



Follow standard procedures for implantation.

02 Confirmation of oral condition



Through full mouth radiograph, confirm the condition of the jawbone.

03 Install the keeper



Screw the keeper/abutment ring into the fixture.

Note: Please use the MAGFIT IP specific driver at the recommended torque setting.

04 Denture design



Place the magnetic assembly on the keeper, take the impression, and fabricate the working denture model.

Denture design should be carried out in the usual manner.

05 Denture making



Denture fabrication is done according to standard procedure.

Note: Please keep enough space in the denture base for the magnetic assembly.

06 Magnetic assembly pick-up



Fabricate a hole into the denture base.

Through the hole, apply adhesive resin carefully between the magnetic assembly and the denture taking care not to move the magnetic assembly out of alignment.

07 Magnetic assembly cementing



After lifting the denture base with magnetic assembly out of the oral cavity, cement the magnetic assembly completely. Note: Please block out excess resin with use of impression material.

08 Completion of denture making



Completion of the final magnet-denture.

09 Fitting of the denture



Installed finished magnet-denture is shown.